


U-500 KwikPen® Quick Reference Guide

BEFORE YOU START

- Required SLM must be completed prior to use
- Do NOT use on more than one patient
- Always use a new needle

SET UP

1. Remove pen cap (do NOT remove label)
2. Clean rubber seal with alcohol swab
3. Attach new needle securely
4. Remove outer needle shield (keep), then inner shield (discard)

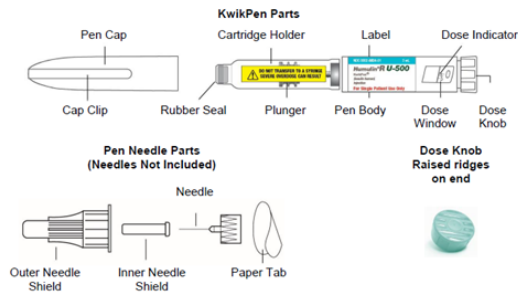

PRIMING (BEFORE EACH INJECTION)

- Turn dose knob to 5 units
- Hold pen needle-up and tap gently
- Press dose knob until "0" appears; count to 5
- Insulin should appear at needle tip



ADMINISTRATION: Turn the dial to ordered dose and push plunger and inject at 90-degree angle to administer in appropriate SQ location. Hold for 10 secs. Recap needle and remove needle.


KEY POINTS

- Each click = 5 units
- Even doses shown as numbers; odd doses shown as lines
- Dose resets to zero after injection
- Small air bubbles are normal

Visual Reference for set up and priming:

Step 1:

- Pull the Pen Cap straight off.
 - Do not remove the KwikPen Label.
 - Wipe the Rubber Seal with an alcohol swab.
- HUMULIN R U-500 should look clear and colorless. Do not use if it is cloudy, colored, or has particles or clumps in it.

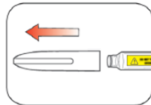


Step 2:

- Select a new Needle.
- Pull off the Paper Tab from the Outer Needle Shield.




Step 3:

- Push the capped Needle straight onto the Pen and twist the Needle on until it is tight.

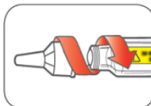

Step 4:

- Pull off the Outer Needle Shield. Do not throw it away.
- Pull off the Inner Needle Shield and throw it away.

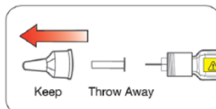


Priming your Pen

Prime before each injection.

- Priming your Pen means removing the air from the Needle and Cartridge that may collect during normal use and ensures that the Pen is working correctly.
- If you do not prime before each injection, you may get too much or too little insulin.

Step 5:

- To prime your pen, turn the Dose Knob to select 5 units.

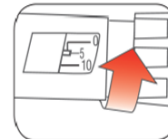

Step 6:

- Hold your Pen with the Needle pointing up. Tap the Cartridge Holder gently to collect air bubbles at the top.



Step 7:

- Continue holding your Pen with Needle pointing up. Push the Dose Knob in until it stops, and "0" is seen in the Dose Window. Hold the Dose Knob in and count to 5 slowly.

You should see insulin at the tip of the Needle.

- If you do not see insulin, repeat priming steps 5 to 7, no more than 8 times.
- If you still do not see insulin, change the Needle and repeat priming steps 5 to 7.

Small air bubbles are normal and will not affect your dose.

